
ILUMARK

an IZI Medical Company

ILUMARK™ Navigation Marker - Twist (REF. 2020)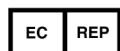

ILUMARK GmbH
Hohenlindner Str. 11c
85622 Feldkirchen
Germany
Tel: +49 (0)89 907 795 98-0
Fax: +49 (0)89 907 795 98-0
Email: info@ilumark.com
www.ilumark.com

CASCINATION AG
Steigerhübelstrasse 3
3008 Bern, Bern
Switzerland

121Medical Products, LLC.
5 Easter Court, Suite J
Owings Mills, MD 21117
USA

Do not use if package is damaged
Nepoužívejte, pokud je poškozený obal
Ne pas utiliser/en cas d'emballage endommagé
Non utilizzare in caso di confezione danneggiata
No utilizari en caso de estadiañado

Store in dark place
Skladujte mimo dosah slunečního záření
Conservare l'abide la lumiere du soleil
Proteggere dall'esposizione alla luce debole
Protegerde la luz debole/durante su almacenamiento

Store in dry place
Skladujte na suchém místě
Conservervdans un lieucec
Conservare in luogo asciutto
Conservaren en lugarseco

Use by- month&year
Použitelné až do měsíce/rok
Antifessivant le mois/ann(OO)
Dausarsi entro mese anno
Cadauca-mes yaflo

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This product meets the essential requirements of Medical Device Directive 93/42 EEC

ENGLISH

User Instructions for ILUMARK™ Navigation Marker - Twist

Intended use
ILUMARK™ Navigation Markers are intended to be used as image guided surgery system accessory to aid in auto-registration and localization of rigid patient anatomical structure in either open or percutaneous image guided surgery procedures. It is indicated for any medical condition in which the use of an image guided surgery system may be considered to be appropriate.

ILUMARK™ Navigation Markers can be used in following procedures:
Stereotactic brain surgery
Endoscopic Sinus Surgery
Pedicule Screw Placement
Reconstructive orthopedic surgery
Soft tissue treatment

⚠ Other applications were not considered and are subject to the responsibility of the user.

🔴 ILUMARK™ Navigation Markers - Twist are packed in blue blister packaging. Use these markers only on M3.

Compatibility
ILUMARK™ Navigation Markers - Twist are compatible with BrainLAB's®:
VectorVision®
Kolibri™

"POINT" Kinguide Agile Hybrid Navigation System
"POINT" Kinguide Robotic-Assisted Surgical System
and CASCINATION®

Instruction for use
Take the ILUMARK™ Navigation Marker - Twist out of the inner sterile packaging with sterile, dry and clean gloves and gently place the hole of the sphere over the threaded pin. Screw the sphere onto the pin until there is no visible gap between the bottom side of the sphere and the base of the pin.

⚠ Do not use a ILUMARK™ Navigation Marker - Twist, if it is not securely placed in the correct position as shown above.

Only use clean and dry ILUMARK™ Navigation Markers - Twist. If the surface of a ILUMARK™ Navigation Marker - Twist is soiled or wet, navigation will be inaccurate. To ensure accurate navigation, wet or soiled ILUMARK™ Navigation Markers - Twist must either be cleaned or replaced with new ILUMARK™ Navigation Markers - Twist. Only use a soft cloth moistened with sterile water to clean the surface of soiled ILUMARK™ Navigation Markers - Twist. Thoroughly dry ILUMARK™ Navigation Markers - Twist before use.

⚠ If ILUMARK™ Navigation Markers - Twist are used on vibrating instruments, regularly verify that they are securely attached.

⚠ If the sterile packaging is damaged or sterility is in doubt, do not use the ILUMARK™ Navigation Markers - Twist, but dispose of them.

🔴 Only physicians experienced with Image Guided Surgery Systems should treat patients with ILUMARK™ Navigation Markers.

Handling
Disposable retro-reflective ILUMARK™ Navigation Markers - Twist are highly sensitive medical devices. They should be handled with care and not dropped. Do not use defective or deformed ILUMARK™ Navigation Markers - Twist.

Storage
Products must be stored in the original unopened packaging in a dry place.

⚠ Do not re-sterilize ILUMARK™ Navigation Markers - Twist!

⚠ Re-sterilization may damage the surface or geometry of the marker sphere. Damages may not be visible to the human eye, but will result in inaccurate navigation and possible patient injuries.

For further information, please contact ILUMARK™ customer support.

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